

CLEARTEST® DIAGNOSTIK

CLEARTEST® Multi Drug 6


A rapid test for the simultaneous, qualitative detection of multiple drugs and drug metabolites in human urine.
For healthcare professionals including professionals at point of care sites.

Immunoassay for in vitro diagnostic use only.

INSTRUCTIONS FOR USE



Instructions for use for testing any combination of the following 6 drugs: AMP/BZO/COC/THC/MOP or MET/MTD.

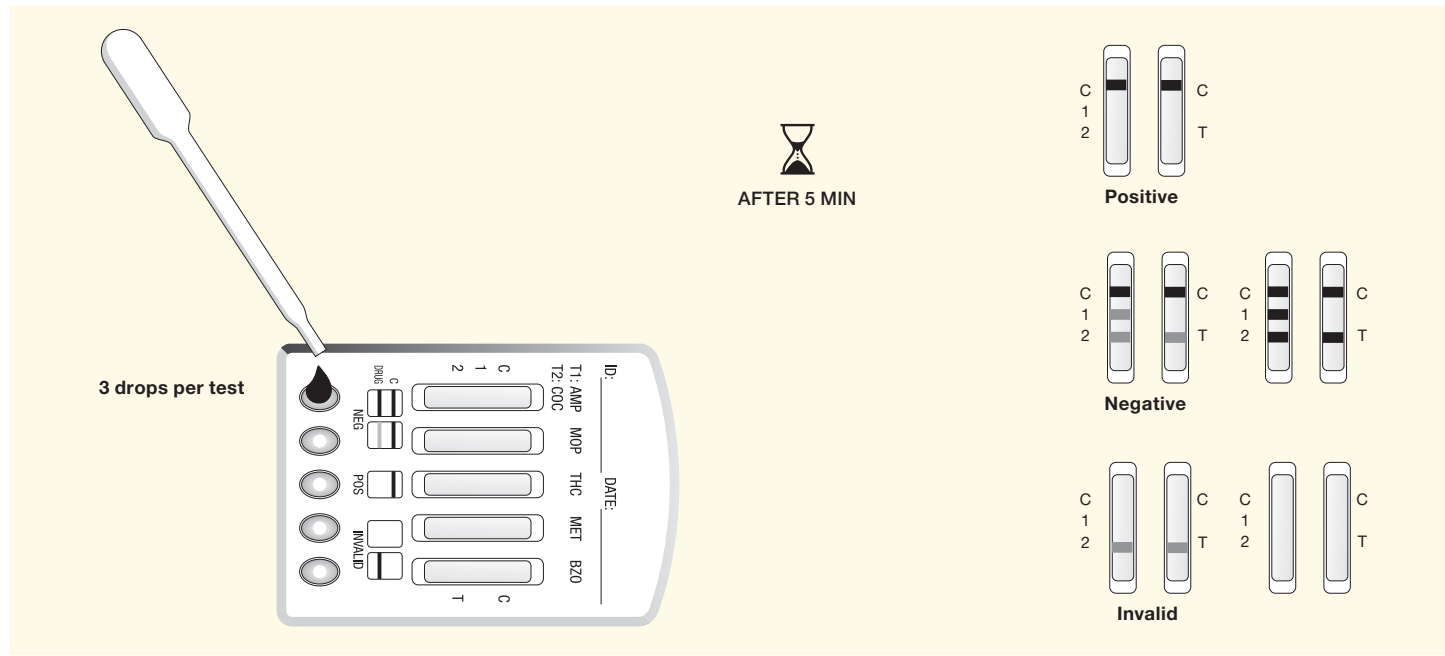

INTENDED USE

The CLEARTEST® Multi Drug 6 is a rapid chromatographic immunoassay for the qualitative detection of multiple drugs and drug metabolites in urine at the following cut-off concentrations:

Test	Calibrator	Cut-off (ng/mL)
Amphetamine (AMP1000)	d-Amphetamine	1.000
Benzodiazepines (BZO 300)	Oxazepam	300
Cocaine (COC 300)	Benzoylcegonine	300
Marijuana (THC50)	11-nor- Δ^9 -THC-9 COOH	50
Methamphetamine (MET1000)	D-Methamphetamin	1.000
Methadone (MTD 300)	Methadone	300
Morphine (MOP 300)	Morphine	300

Configurations of the CLEARTEST® Multi Drug 6 come with any combination of the above listed drug analytes with or without S.V.T. This assay provides only a preliminary analytical test result. A more specific alternate chemical method must be used in order to obtain a confirmed analytical result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method. Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are indicated.

SUMMARY

The CLEARTEST® Multi Drug 6 is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes monoclonal antibodies to selectively detect elevated levels of specific drugs in urine.

AMPHETAMINE (AMP)

Amphetamine is a Schedule II controlled substance available by prescription (Dexedrine®) and is also available on the illicit market. Amphetamines are a class of potent sympathomimetic agents with therapeutic applications. They are chemically related to the human body's natural catecholamines: epinephrine and norepinephrine. Acute higher doses lead to enhanced stimulation of the central nervous system (CNS) and induce euphoria, alertness, reduced appetite, and a sense of increased energy and power. Cardiovascular responses to amphetamines include increased blood pressure and cardiac arrhythmias. More acute responses produce anxiety, paranoia, hallucinations, and psychotic behavior. The effects of Amphetamines generally last 2-4 hours following use and the drug has a half-life of 4-24 hours in the body. About 30% of amphetamines are excreted in the urine in unchanged form, with the remainder as hydroxylated and deaminated derivatives. The CLEARTEST® Multi Drug 6 yields a positive result when the concentration of amphetamines in urine exceeds detective level.

BENZODIAZEPINES (BZO)

Benzodiazepines are medications that are frequently prescribed for the symptomatic treatment of anxiety and sleep disorders. They produce their effects via specific receptors involving a neurochemical called gamma aminobutyric acid (GABA). Because they are safer and more effective, benzodiazepines have replaced barbiturates in the treatment of both anxiety and insomnia. Benzodiazepines are also used as sedatives before some surgical and medical procedures, and for the treatment of seizure disorders and alcohol withdrawal. Risk of physical dependence increases if benzodiazepines are taken regularly (e.g., daily) for more than a few months, especially at higher than normal doses. Stopping abruptly can bring on such symptoms as trouble sleeping, gastrointestinal upset, feeling unwell, loss of appetite, sweating, trembling, weakness, anxiety and changes in perception.

Only trace amounts (less than 1%) of most benzodiazepines are excreted unaltered in the urine; most of the concentration in urine is conjugated drug. The detection period for benzodiazepines in urine is 3 - 7 days. The CLEARTEST® Multi Drug 6 yields a positive result when the concentration of benzodiazepines in urine exceeds detective level.

COCAINE (COC)

Cocaine is a potent central nervous system stimulant and a local anesthetic. Initially, it brings about extreme energy and restlessness while gradually resulting in tremors, over-sensitivity and spasms. In large amounts, cocaine causes fever, unresponsiveness, difficulty in breathing and unconsciousness.

Cocaine is often self-administered by nasal inhalation, intravenous injection and free-base smoking. It is excreted in the urine in a short time primarily as benzoylecgonine.^{3,4} Benzoylecgonine, a major metabolite of cocaine, has a longer biological half-life (5-8 hours) than cocaine (0.5-1.5 hours), and can generally be detected for 24-48 hours after cocaine exposure.⁴

The CLEARTEST® Multi Drug 6 yields a positive result when the concentration of Cocaine in urine exceeds detective level.

MARIJUANA (THC)

THC (Δ^9 -tetrahydrocannabinol) is the primary active ingredient in cannabis (marijuana). When smoked or orally administered, THC produces euphoric effects. Users have impaired short-term memory and slowed learning. They may also experience transient episodes of confusion and anxiety. Long-term, relatively heavy use may be associated with behavioral disorders. The peak effect of marijuana administered by smoking occurs in 20-30 minutes and the duration is 90-120 minutes after one cigarette. Elevated levels of urinary metabolites are found within hours of exposure and remain detectable for 3-10 days after smoking. The main metabolite excreted in the urine is 11-nor- Δ^9 -tetrahydrocannabinol-9-carboxylic acid (THC-COOH).

The CLEARTEST® Multi Drug 6 yields a positive result when the concentration of THC COOH in urine exceeds detective level.

METHAMPHETAMINE (MET)

Methamphetamine is an addictive stimulant drug that strongly activates certain systems in the brain. Methamphetamine is closely related chemically to Amphetamine, but the central nervous system effects of Methamphetamine are greater. Methamphetamine is made in illegal laboratories and has a high potential for abuse and dependence. The drug can be taken orally, injected, or inhaled. Acute higher doses lead to enhanced stimulation of the central nervous system and induce euphoria, alertness, reduced appetite, and a sense of increased energy and power. Cardiovascular responses to Methamphetamine include increased blood pressure and cardiac arrhythmias. More acute responses produce anxiety, paranoia, hallucinations, psychotic behavior, and eventually, depression and exhaustion.

The effects of Methamphetamine generally last 2-4 hours and the drug have a half-life of 9-24 hours in the body. Methamphetamine is excreted in the urine primarily as Amphetamine, and oxidized and deaminated derivatives. However, 10-20% of Methamphetamine is excreted unchanged. Thus, the presence of the parent compound in the urine indicates Methamphetamine use. Methamphetamine is generally detectable in the urine for 3-5 days, depending on urine pH level.

The CLEARTEST® Multi Drug 6 is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of Methamphetamine in urine. The CLEARTEST® Multi Drug 6 yields a positive result when the Methamphetamine in urine exceeds detective level.

METHADONE (MTD)

Methadone is a narcotic analgesic prescribed for the management of moderate to severe pain and for the treatment of opiate dependence (heroin, Vicodin, Percocet, morphine). The pharmacology of oral methadone is very different from IV methadone. Oral methadone is partially stored in the liver for later use. IV methadone acts more like heroin. In most states you must go to a pain clinic or a methadone maintenance clinic to be prescribed methadone.

Methadone is a long acting pain reliever producing effects that last from twelve to forty-eight hours. Ideally, methadone frees the client from the pressures of obtaining illegal heroin, from the dangers of injection, and from the emotional roller coaster that most opiates produce. Methadone, if taken for long periods and at large doses, can lead to a very long withdrawal period. The withdrawals from methadone are more prolonged and troublesome than those provoked by heroin cessation, yet the substitution and phased removal of methadone is an acceptable method of detoxification for patients and therapists.⁷

The CLEARTEST® Multi Drug 6 yields a positive result when the concentration of methadone in urine exceeds detective level.

MORPHINE (MOP)/OPIATE (OPI)

Opiate refers to any drug that is derived from the opium poppy, including the natural products, morphine and codeine, and the semi-synthetic drugs such as heroin. Opioid is more general, referring to any drug that acts on the opioid receptor.

Opioid analgesics comprise a large group of substances which control pain by depressing the CNS. Large doses of morphine can produce higher tolerance levels, physiological dependency in users, and may lead to substance abuse. Morphine is excreted unmetabolized, and is also the major metabolic product of codeine and heroin. Morphine is detectable in the urine for several days after an opiate dose.²

The CLEARTEST® Multi Drug 6 yields a positive result when the concentration of morphine/Opiate in urine exceeds detective level.

PRINCIPLE (FOR DOA TESTS EXCLUDING ALCOHOL)

During testing, a urine specimen migrates upward by capillary action. A drug, if present in the urine specimen below its cut-off concentration, will not saturate the binding sites of its specific antibody. The antibody will then react with the drug-protein conjugate and a visible colored line will show up in the test region of the specific drug dipstick. The presence of drug above the cut-off concentration will saturate all the binding sites of the antibody. Therefore, the colored line will not form in the test region.

A drug-positive urine specimen will not generate a colored line in the specific test region of the dipstick because of drug competition, while a drug-negative urine specimen will generate a line in the test region because of the absence of drug competition.

To serve as a procedural control, a colored line will always appear at the control region, indicating that proper volume of specimen has been added and membrane wicking has occurred.

REAGENTS (FOR DOA TESTS EXCLUDING ALCOHOL)

Each test line contains anti-drug mouse monoclonal antibody and corresponding drug-protein conjugates. The control line contains goat anti-rabbit IgG polyclonal antibodies and rabbit IgG.

PRECAUTIONS

- For healthcare professionals including professionals at point of care sites.
- Immunoassay for *in vitro* diagnostic use only. The Test Panel should remain in the sealed pouch until use.
- All specimens should be considered potentially hazardous and handled in the same manner as an infectious agent.
- The used Test cassette should be discarded according to local regulations.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable through the expiration date printed on the sealed pouch. The Test cassette must remain in the sealed pouch until use.

DO NOT FREEZE.

Do not use beyond the expiration date.

SPECIMEN COLLECTION AND PREPARATION

Urine Assay

The urine specimen should be collected in a clean and dry container. Urine collected at any time of the day may be used. Urine specimens exhibiting visible precipitates should be centrifuged, filtered, or allowed to settle to obtain a clear specimen for testing.

Specimen Storage

Urine specimens may be stored at 2-8°C for up to 48 hours prior to testing. For prolonged storage, specimens may be frozen and stored below -20°C. Frozen specimens should be thawed and mixed well before testing. When testing cards with S.V.T. or Alcohol storage of urine specimens should not exceed 2 hours at room temperature or 4 hours refrigerated prior to testing.

MATERIALS

Materials Provided

- Test cassettes
- Instructions for use
- Adulteration Color Chart (when applicable)

Materials Required But Not Provided

- Timer


DIRECTIONS FOR USE

Allow the test, urine specimen, and/or controls to reach room temperature (15-30°C) prior to testing.

1. Bring the pouch to room temperature before opening it. Remove the test cassette from the sealed pouch and use it within one hour.
2. Place the test cassette on a clean and level surface. Hold the dropper vertically and transfer 3 full drops of urine (approx. 75 mL) to the specimen well (S) of the test cassette, and then start the timer. Avoid trapping air bubbles in the specimen well (S). See the illustration above.
3. Read the adulteration strips and Alcohol strip between 3-5 minutes according to color chart provided separately/on foil pouch. Refer to your Drug Free Policy for guidelines on adulterated specimens. We recommend not to interpret the drug test results and either retest the urine or collect another specimen in case of any positive result for any adulteration test.
4. The drug strip result should be read at 5 minutes. Do not interpret the result after 10 minutes.


INTERPRETATION OF RESULTS

NEGATIVE* A colored line appears in the Control region (C) and colored lines appear in the Test region (T). This negative result means that the concentrations in the urine sample are below the designated cut-off levels for a particular drug tested.




*NOTE: The shade of the colored lines(s) in the Test region (T) may vary. The result should be considered negative whenever there is even a faint line.

POSITIVE A colored line appears in the Control region (C) and NO line appears in the Test region (T). The positive result means that the drug concentration in the urine sample is greater than the designated cut-off for a specific drug.



INVALID No line appears in the Control region (C). Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for Control line failure. Read the directions again and repeat the test with a new test card. If the result is still invalid, contact your manufacturer.

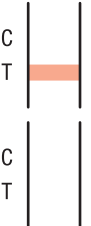

QUALITY CONTROL

A procedural control is included in the test. A line appearing in the control region (C) is considered an internal procedural control. It confirms sufficient specimen volume, adequate membrane wicking and correct procedural technique.

Control standards are not supplied with this kit. However, it is recommended that positive and negative controls be tested as good laboratory practice to confirm the test procedure and to verify proper test performance.

LIMITATIONS

1. The CLEARTEST® Multi Drug 6 provides only a qualitative, preliminary analytical result. A secondary analytical method must be used to obtain a confirmed result. Gas chromatography /mass spectrometry (GC/MS) is the preferred confirmatory method.^{1,10}
2. There is a possibility that technical or procedural errors, as well as interfering substances in the urine specimen may cause erroneous results.
3. Adulterants, such as bleach and/or alum, in urine specimens may produce erroneous results regardless of the analytical method used. If adulteration is suspected, the test should be repeated with another urine specimen.
4. A positive result does not indicate level or intoxication, administration route or concentration in urine.
5. A negative result may not necessarily indicate drug-free urine. Negative results can be obtained when drug is present but below the cut-off level of the test.
6. This test does not distinguish between drugs of abuse and certain medications.
7. A positive test result may be obtained from certain foods or food supplements. Alcohol in the atmosphere, such as spray from perfumes, deodorizers, glass cleaners etc. can affect the Alcohol Rapid Tests. Therefore, adequate measures should be taken to avoid undue interference from such atmospheric agents in the testing area.
8. The test is only for detection of presence/ absence of alcohol in the urine, which may result from habitual drinking or medications and does not discriminate the two.

EXPECTED VALUES

The negative result indicates that the drug concentration is below the detectable level. Positive result means the concentration of drug is above the detectable level.

PERFORMANCE CHARACTERISTICS

Accuracy

A side-by-side comparison was conducted using the CLEARTEST® Multi Drug 6 and commercially available drug rapid tests. Testing was performed on approximately hundred specimens per drug type previously collected from subjects presenting for Drug Screen Testing. Presumptive positive results were confirmed by GC/MS.

Method	GC/MS		% agreement with GC/MS	
	Positive	Negative		
CLEARTEST® Multi Drug 6	Positive	103	3	98.1%
	Negative	2	142	97.9%
AMP 1,000	Positive	110	2	99.1%
	Negative	1	137	98.6%
AMP 500	Positive	116	2	99.1%
	Negative	1	131	98.5%
AMP 300	Positive	98	2	96.1%
	Negative	4	146	98.6%
BAR 300	Positive	101	3	95.3%
	Negative	5	141	97.9%
BAR 200	Positive	112	3	98.2%
	Negative	2	133	97.8%
BZO 500	Positive	121	1	98.4%
	Negative	2	126	99.2%
BZO 300	Positive	127	2	99.2%
	Negative	1	120	98.4%

Method		GC/MS		% agreement with GC/MS
CLEARTEST® Multi Drug 6		Positive	Negative	
BZO 100	Positive	128	3	99.2%
	Negative	1	118	97.5%
BUP 50	Positive	105	0	99.1%
	Negative	1	144	>99.9%
COC 300	Positive	111	3	98.2%
	Negative	2	134	97.8%
COC 100	Positive	117	4	99.2%
	Negative	1	128	97.0%
THC 150	Positive	86	4	94.5%
	Negative	5	155	97.5%
THC 50	Positive	92	3	97.9%
	Negative	2	153	98.1%
THC 25	Positive	95	4	96.9%
	Negative	3	148	97.4%
MTD 300	Positive	89	2	98.9%
	Negative	1	158	98.8%
MTD 200	Positive	91	2	98.7%
	Negative	1	156	98.7%
MET 1,000	Positive	76	5	96.2%
	Negative	3	166	97.1%
MET 500	Positive	83	5	97.6%
	Negative	2	160	97.0%
MET 300	Positive	88	4	97.8%
	Negative	2	156	97.5%
MDMA 1,000	Positive	99	1	98.0%
	Negative	2	148	99.3%
MDMA 500	Positive	102	1	98.1%
	Negative	2	145	99.3%
MOP 300	Positive	95	7	95.0%
	Negative	5	143	95.3%
MOP 100	Positive	98	5	97.0%
	Negative	3	144	96.6%
MQL	Positive	79	11	89.8%
	Negative	9	151	93.2%
OPI	Positive	117	8	96.7%
	Negative	4	121	93.8%
PCP	Positive	85	5	92.4%
	Negative	7	153	96.8%
PPX	Positive	97	9	96.0%
	Negative	4	140	94.0%
TCA	Positive	91	13	94.8%
	Negative	5	141	91.6%

Method		GC/MS		% agreement with GC/MS
CLEARTEST® Multi Drug 6		Positive	Negative	
TML	Positive	82	12	88.2%
	Negative	11	145	92.4%
KET 1,000	Positive	77	3	97.5%
	Negative	2	168	98.2%
KET 500	Positive	81	3	97.6%
	Negative	2	164	98.2%
KET 300	Positive	89	4	96.7%
	Negative	3	154	97.5%
KET 100	Positive	97	4	96.0%
	Negative	4	145	97.3%
OXY 100	Positive	84	1	97.7%
	Negative	2	163	99.4%
COT 200	Positive	88	4	96.7%
	Negative	3	155	97.5%
COT 100	Positive	93	3	97.9%
	Negative	2	152	98.1%
EDDP 300	Positive	92	1	97.9%
	Negative	2	155	99.4%
EDDP 100	Positive	95	5	96.9%
	Negative	3	147	96.7%
FYL 20	Positive	79	1	98.8%
	Negative	1	169	99.4%
FYL 10	Positive	80	1	98.8%
	Negative	1	168	99.4%
K2-50	Positive	78	3	97.5%
	Negative	2	167	98.2%
K2-30	Positive	82	2	97.6%
	Negative	2	164	98.8%
6-MAM10	Positive	42	2	97.7%
	Negative	1	105	98.1%
MDA500	Positive	103	3	98.1%
	Negative	2	142	97.9%

PRECISION

A study was conducted at three hospitals by laypersons using three different lots of product to demonstrate the within run, between run and between operator precision. An identical card of coded specimens, containing drugs at concentrations of 50% and 25% cut-off level, was labeled, blinded and tested at each site. The results are given below:

AMPHETAMINE (AMP 1,000)

Amphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
500	10	10	0	10	0	10	0
750	10	9	1	8	2	9	1
1,250	10	1	9	2	8	2	8
1,500	10	0	10	0	10	0	10

AMPHETAMINE (AMP 500)

Amphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
250	10	10	0	10	0	10	0
375	10	9	1	9	1	9	1
625	10	2	8	1	9	2	8
750	10	0	10	0	10	0	10

AMPHETAMINE (AMP 300)

Amphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	8	2	8	2	8	2
375	10	2	8	2	8	2	8
450	10	0	10	0	10	0	10

BARBITURATES (BAR 300)

% AGREEMENT WITH COMMERCIAL KIT

	AMP 1,000	AMP 500	AMP 300	BAR 300	BAR 200	BZO 500	BZO 300	BZO 200	BZO 100	BUP	COC 300	
Positive Agreement	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	
Negative Agreement	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	
Total Results	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	
	COC 100	THC 150	THC 50	THC 25	MTD 300	MTD 200	MET 1,000	MET 500	MET 300	MDMA 1,000	MDMA 500	
Positive Agreement	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	
Negative Agreement	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	
Total Results	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	>99.9%	
	MOP 300	MOP 100	MQL	OPI	PCP	PPX	TCA	TML	KET 1,000	KET 500	KET 300	
Positive Agreement	>99.9%	>99.9%	>99.9%	*	>99.9%	>99.9%	*	*	>99.9%	>99.9%	>99.9%	
Negative Agreement	>99.9%	>99.9%	>99.9%	*	>99.9%	>99.9%	*	*	>99.9%	>99.9%	>99.9%	
Total Results	>99.9%	>99.9%	>99.9%	*	>99.9%	>99.9%	*	*	>99.9%	>99.9%	>99.9%	
	KET 100	OXY	COT 200	COT 100	EDDP 300	EDDP 100	FYL 20	FYL 10	K2 50	K2 30	6-MAM 10	MDA 500
Positive Agreement	>99.9%	*	*	*	*	*	*	*	*	*	*	*
Negative Agreement	>99.9%	*	*	*	*	*	*	*	*	*	*	*
Total Results	>99.9%	*	*	*	*	*	*	*	*	*	*	*

* Note: Based on GC/MS data instead of Commercial Kit.

Secobarbital conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	9	1	8	2	9	1
375	10	2	8	1	9	2	8
450	10	0	10	0	10	0	10

BARBITURATES (BAR 200)

Secobarbital conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
100	10	10	0	10	0	10	0
150	10	9	1	9	1	9	1
250	10	1	9	1	9	1	9
300	10	0	10	0	10	0	10

BENZODIAZEPINES (BZO 500)

Oxazepam conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
250	10	10	0	10	0	10	0
375	10	8	2	9	1	8	2
625	10	1	9	2	8	1	9
750	10	0	10	0	10	0	10

BENZODIAZEPINES (BZO 300)

Oxazepam conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	9	1	9	1	9	1
375	10	1	9	1	9	1	9
450	10	0	10	0	10	0	10

BENZODIAZEPINES (BZO 200)

Oxazepam conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
100	10	10	0	10	0	10	0
150	10	9	1	8	2	9	1
250	10	1	9	1	9	2	8
300	10	0	10	0	10	0	10

BENZODIAZEPINES (BZO 100)

Oxazepam conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
50	10	10	0	10	0	10	0
75	10	9	1	8	2	7	3
125	10	1	9	1	9	2	8
150	10	0	10	0	10	0	10

BUPRENORPHINE (BUP)

Buprenorphine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
5	10	10	0	10	0	10	0
7.5	10	9	1	9	1	8	2
12.5	10	1	9	1	9	1	9
15	10	0	10	0	10	0	10

COCAINE (COC 300)

Benzoylecgonine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	9	1	9	1	9	1
375	10	1	9	1	9	1	9
450	10	0	10	0	10	0	10

COCAINE (COC 100)

Benzoylecgonine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
50	10	10	0	10	0	10	0
75	10	9	1	9	1	9	1
125	10	2	8	2	8	2	8

150	10	0	10	0	10	0	10
-----	----	---	----	---	----	---	----

MARIJUANA (THC150)

11-nor- Δ^9 -COOH conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
75	10	10	0	10	0	10	0
112.5	10	9	1	9	1	9	1
187.5	10	2	8	1	9	1	9
225	10	0	10	0	10	0	10

MARIJUANA (THC50)

11-nor- Δ^9 -COOH conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
25	10	10	0	10	0	10	0
37.5	10	9	1	8	2	9	1
62.5	10	1	9	1	9	2	8
75	10	0	10	0	10	0	10

MARIJUANA (THC25)

11-nor- Δ^9 -COOH conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
12.5	10	10	0	10	0	10	0
18.75	10	8	2	8	2	8	2
31.25	10	1	9	1	9	2	8
37.5	10	0	10	0	10	0	10

METHADONE (MTD300)

Methadone conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	9	1	9	1	9	1
375	10	1	9	1	9	1	9
450	10	0	10	0	10	0	10

METHADONE (MTD200)

Methadone conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
100	10	10	0	10	0	10	0
150	10	8	2	8	2	8	2
250	10	1	9	1	9	2	8
300	10	0	10	0	10	0	10

METHAMPHETAMINE (MET1,000)

Methamphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
500	10	10	0	10	0	10	0
750	10	9	1	9	1	9	1
1,250	10	1	9	2	8	1	9
1,500	10	0	10	0	10	0	10

METHAMPHETAMINE (MET 500)

Methamphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
250	10	10	0	10	0	10	0
375	10	9	1	9	1	9	1
625	10	1	9	1	9	1	9
750	10	0	10	0	10	0	10

METHAMPHETAMINE (MET300)

Methamphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	9	1	9	1	9	1
375	10	1	9	1	9	1	9
450	10	0	10	0	10	0	10

METHYLENEDIOXYMETHAMPHETAMINE (MDMA1, 000) ECSTASY

Methylenedioxyamphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
500	10	10	0	10	0	10	0
750	10	9	1	9	1	8	2
1,250	10	1	9	1	9	1	9
1,500	10	0	10	0	10	0	10

METHYLENEDIOXYMETHAMPHETAMINE (MDMA 500) ECSTASY

Methylenedioxyamphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
250	10	10	0	10	0	10	0
375	10	8	2	9	1	9	1
625	10	1	9	1	9	1	9
750	10	0	10	0	10	0	10

MORPHINE (MOP 300)

Morphine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0

225	10	9	1	9	1	9	1
375	10	1	9	1	9	1	9
450	10	0	10	0	10	0	10

MORPHINE (MOP 100)

Morphine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
50	10	10	0	10	0	10	0
75	10	9	1	9	1	9	1
125	10	1	9	1	9	1	9
150	10	0	10	0	10	0	10

METHAQUALONE (MQL 300)

Methaqualone conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	9	1	9	1	9	1
375	10	1	9	1	9	1	9
450	10	0	10	0	10	0	10

MORPHINE/OPIATE (OPI 2,000)

Morphine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
1,000	10	10	0	10	0	10	0
1,500	10	9	1	9	1	9	1
2,500	10	1	9	1	9	1	9
3,000	10	0	10	0	10	0	10

PHENCYCLIDINE (PCP)

Phencyclidine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
12.5	10	10	0	10	0	10	0
18.75	10	8	2	9	1	9	1
31.25	10	1	9	1	9	1	9
37.5	10	0	10	0	10	0	10

PROPOXYPHENE (PPX)

Propoxyphene conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	8	2	9	1	9	1
375	10	1	9	1	9	1	9
450	10	0	10	0	10	0	10

TRICYCLIC ANTIDEPRESSANTS (TCA)

Nortriptyline conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
500	10	10	0	10	0	10	0
750	10	9	1	8	2	8	2
1,250	10	1	9	1	9	1	9
1,500	10	0	10	0	10	0	10

TRAMADOL (TML)

Tramadol conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
50	10	10	0	10	0	10	0
75	10	9	1	9	1	8	2

Tramadol conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
125	10	1	9	1	9	2	8
150	10	0	10	0	10	0	10

KETAMINE (KET1, 000)

Ketamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
500	10	10	0	10	0	10	0
750	10	9	1	8	2	9	1
1,250	10	1	9	1	9	2	8
1,500	10	0	10	0	10	0	10

KETAMINE (KET500)

Ketamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
250	10	10	0	10	0	10	0
375	10	9	1	9	1	8	2
625	10	1	9	1	9	2	8
750	10	0	10	0	10	0	10

KETAMINE (KET300)

Ketamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	9	1	9	1	9	1
375	10	1	9	1	9	1	9
450	10	0	10	0	10	0	10

KETAMINE (KET100)

Ketamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
50	10	10	0	10	0	10	0
75	10	9	1	9	1	9	1
125	10	1	9	1	9	2	8
150	10	0	10	0	10	0	10

OXYCODONE (OXY100)

Oxycodone conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
50	10	10	0	10	0	10	0
75	10	9	1	9	1	9	1
125	10	1	9	1	9	1	9
150	10	0	10	0	10	0	10

COTININE (COT 200)

Cotinine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
100	10	10	0	10	0	10	0
150	10	9	1	9	1	9	1
250	10	1	9	1	9	2	8
300	10	0	10	0	10	0	10

COTININE (COT 100)

Cotinine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
50	10	10	0	10	0	10	0
75	10	9	1	9	1	9	1
125	10	1	9	1	9	1	9
150	10	0	10	0	10	0	10

2-ETHYLIDENE-1,5-DIMETHYL-3,3-DIPHENYLPYRROLIDINE (EDDP 300)

EDDP conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
150	10	10	0	10	0	10	0
225	10	9	1	9	1	9	1
375	10	1	9	2	8	1	9
450	10	0	10	0	10	0	10

2-ETHYLIDENE-1,5-DIMETHYL-3,3-DIPHENYLPYRROLIDINE (EDDP 100)

EDDP conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
50	10	10	0	10	0	10	0
75	10	9	1	9	1	9	1
125	10	1	9	1	9	1	9
150	10	0	10	0	10	0	10

FENTANYL (FYL20)

FYL conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
10	10	10	0	10	0	10	0
15	10	9	1	9	1	9	1
25	10	1	9	1	9	1	9
30	10	0	10	0	10	0	10

FENTANYL (FYL10)

FYL conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
5	10	10	0	10	0	10	0
7.5	10	9	1	9	1	9	1

12.5	10	1	9	1	9	1	9
15	10	0	10	0	10	0	10

K2 50

K2 conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
25	10	10	0	10	0	10	0
37.5	10	8	2	8	2	9	1
62.5	10	1	9	2	8	2	8
75	10	0	10	0	10	0	10

K2 30

K2 conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
15	10	10	0	10	0	10	0
22.5	10	8	2	9	1	9	1
37.5	10	1	9	1	9	1	9
45	10	0	10	0	10	0	10

6-MAM

6-MAM conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
5	10	10	0	10	0	10	0
7.5	10	9	1	9	1	9	1
12.5	10	1	9	1	9	1	9
15	10	0	10	0	10	0	10

MDA 500

MDA conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	10	10	0	10	0	10	0
250	10	10	0	10	0	10	0
375	10	9	1	9	1	9	1
625	10	1	9	1	9	1	9
750	10	0	10	0	10	0	10

ANALYTISCHE SENSITIVITÄT

Eine drogenfreie Urinprobe wurde mit Drogen in den aufgelisteten Konzentrationen versetzt. Die Ergebnisse sind im Folgenden zusammengefasst:

Drug Concentration Cut-off Range	AMP1,000		AMP500		AMP 300		BAR 300		BAR 200		BZO500		BZO300		BZO200	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-25% Cut-off	26	4	25	5	27	3	27	3	26	4	27	3	27	3	27	3
Cut-off	15	15	15	15	15	15	16	14	15	15	15	15	15	15	16	14
+25% Cut-off	3	27	3	27	4	26	4	26	3	27	4	26	3	27	3	27
+50% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	BZO100		BUP		COC300		COC100		THC150		THC50		THC25		MTD300	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-25% Cut-off	27	3	26	4	26	4	27	3	27	3	26	4	27	3	26	4
Cut-off	14	16	14	16	13	17	16	14	15	15	14	16	15	15	14	16
+25% Cut-off	3	27	3	27	3	27	4	26	4	26	3	27	4	26	3	27
+50% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	MTD200		MET1,000		MET500		MET300		MDMA1,000		MDMA500		MOP300		MOP100	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-25% Cut-off	25	5	27	3	27	3	27	3	26	4	25	5	27	3	26	4
Cut-off	15	15	14	16	16	14	15	15	15	15	14	16	15	15	15	15
+25% Cut-off	4	26	3	27	4	26	3	27	5	25	4	26	5	25	3	27
+50% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	OPI		PCP		PPX		TCA		TML		KET1,000		KET500		KET300	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-25% Cut-off	27	3	25	5	26	4	25	5	27	3	27	3	27	3	26	4
Cut-off	14	16	15	15	15	15	15	15	15	15	15	15	15	15	16	14
+25% Cut-off	4	26	3	27	3	27	4	26	4	26	3	27	4	26	4	26
+50% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	KET100		MQL		OXY		COT200		COT100		EDDP300		EDDP100		FYL20	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-25% Cut-off	27	3	27	3	27	3	27	3	27	3	27	3	26	4	27	3
Cut-off	15	15	15	15	15	15	15	15	14	16	15	15	15	15	14	16
+25% Cut-off	3	27	4	26	4	26	4	26	4	26	4	26	3	27	4	26

Drug Concentration Cut-off Range	KET100		MQL		OXY		COT200		COT100		EDDP300		EDDP100		FYL20	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
+50% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	FYL10		K2 50		K2 30		6-MAM 10		MDA 500	
	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0
-25% Cut-off	27	3	27	3	27	3	27	3	26	4
Cut-off	15	15	15	15	16	14	15	15	15	15
+25% Cut-off	3	27	3	27	4	26	4	26	3	27
+50% Cut-off	0	30	0	30	0	30	0	30	0	30
+300% Cut-off	0	30	0	30	0	30	0	30	0	30

ANALYTICAL SPECIFICITY

The following table lists the concentrations of compounds (ng/mL) that are detected as positive in urine by the CLEARTEST® Multi Drug 6 at 5 minutes.

Analytes	Concentration (ng/mL)	Analytes	Concentration (ng/mL)
AMPHETAMINE (AMP 1,000)			
D,L-Amphetamine sulfate	300	Phentermine	1,000
L-Amphetamine	25,000	Maprotiline	50,000
(±) 3,4-Methylenedioxy amphetamine	500	Methoxyphenamine	6,000
		D-Amphetamine	1,000
AMPHETAMINE (AMP 500)			
D,L-Amphetamine sulfate	150	Phentermine	500
L-Amphetamine	12,500	Maprotiline	25,000
(±) 3,4-Methylenedioxy amphetamine	250	Methoxyphenamine	3,000
		D-Amphetamine	500
AMPHETAMINE (AMP 300)			
D,L-Amphetamine sulfate	75	Phentermine	300
L-Amphetamine	10,000	Maprotiline	15,000
(±) 3,4-Methylenedioxy amphetamine	150	Methoxyphenamine	2,000
		D-Amphetamine	300
BARBITURATES (BAR 300)			
Amobarbital	5,000	Alphenol	600
5,5-Diphenylhydantoin	8,000	Aprobarbital	500
Allobarbital	600	Butabarbital	200
Barbital	8,000	Butalbital	8,000
Talbutal	200	Butethal	500
Cyclopentobarbital	30,000	Phenobarbital	300
Pentobarbital	8,000	Secobarbital	300
BARBITURATES (BAR 200)			
Amobarbital	3,000	Alphenol	400
5,5-Diphenylhydantoin	5,000	Aprobarbital	300
Allobarbital	400	Butabarbital	150
Barbital	5,000	Butalbital	5,000
Talbutal	150	Butethal	300
Cyclopentobarbital	20,000	Phenobarbital	200
Pentobarbital	5,000	Secobarbital	200
BENZODIAZEPINES (BZO 500)			
Alprazolam	200	Bromazepam	1,500
a-hydroxyalprazolam	2,500	Chlordiazepoxide	1,500
Clobazam	300	Nitrazepam	300
Clonazepam	800	Norchlordiazepoxide	200
Clorazepate dipotassium	800	Nordiazepam	1,500
Delorazepam	1,500	Oxazepam	500
Desalkylflurazepam	300	Temazepam	300
Flunitrazepam	300	Diazepam	500
(±) Lorazepam	5,000	Estazolam	10,000
RS-Lorazepam glucuronide	300	Triazolam	5,000
Midazolam	10,000		
BENZODIAZEPINES (BZO 300)			
Alprazolam	100	Bromazepam	900
a-hydroxyalprazolam	1,500	Chlordiazepoxide	900
Clobazam	200	Nitrazepam	200
Clonazepam	500	Norchlordiazepoxide	100
Clorazepate dipotassium	500	Nordiazepam	900
Delorazepam	900	Oxazepam	300
Desalkylflurazepam	200	Temazepam	100

Analytes	Concentration (ng/mL)	Analytes	Concentration (ng/mL)
Flunitrazepam	200	Diazepam	300
(±) Lorazepam	3,000	Estazolam	6,000
RS-Lorazepam glucuronide	200	Triazolam	3,000
Midazolam	6,000		
BENZODIAZEPINES (BZO 200)			
Alprazolam	70	Bromazepam	600
a-hydroxyalprazolam	1,000	Chlordiazepoxide	600
Clobazam	120	Nitrazepam	120
Clonazepam	300	Norchlordiazepoxide	70
Clorazepate dipotassium	300	Nordiazepam	600
Delorazepam	600	Oxazepam	200
Desalkylflurazepam	120	Temazepam	70
Flunitrazepam	120	Diazepam	200
(±) Lorazepam	2,000	Estazolam	4,000
RS-Lorazepam glucuronide	120	Triazolam	2,000
Midazolam	4,000		
BENZODIAZEPINES (BZO 100)			
Alprazolam	40	Bromazepam	300
a-hydroxyalprazolam	500	Chlordiazepoxide	300
Clobazam	60	Nitrazepam	60
Clonazepam	150	Norchlordiazepoxide	40
Clorazepate dipotassium	150	Nordiazepam	300
Delorazepam	300	Oxazepam	100
Desalkylflurazepam	60	Temazepam	40
Flunitrazepam	60	Diazepam	100
(±) Lorazepam	1,000	Estazolam	2,000
RS-Lorazepam glucuronide	60	Triazolam	1,000
Midazolam	2,000		
BUPRENORPHINE (BUP)			
Buprenorphine	10	Norbuprenorphine	50
Buprenorphine 3-D-Glucuronide	50	Norbuprenorphine 3-D-Glucuronide	100
COCAINE (COC 300)			
Benzoyllecgonine	300	Cocaethylene	20,000
Cocaine HCl	200	Ecgonine	30,000
COCAINE (COC 100)			
Benzoyllecgonine	100	Cocaethylene	7,000
Cocaine HCl	80	Ecgonine	10,000
MARIJUANA (THC150)			
Cannabinol	100,000	Δ8-THC	50,000
11-nor-Δ8-THC-9 COOH	100	Δ9-THC	50,000
11-nor-Δ9-THC-9 COOH	150		
MARIJUANA (THC50)			
Cannabinol	35,000	Δ8-THC	17,000
11-nor-Δ8-THC-9 COOH	30	Δ9-THC	17,000
11-nor-Δ9-THC-9 COOH	50		
MARIJUANA (THC25)			
Cannabinol	17,500	Δ8-THC	8,500
11-nor-Δ8-THC-9 COOH	15	Δ9-THC	8,500
11-nor-Δ9-THC-9 COOH	25		
METHADONE (MTD300)			
Methadone	300	Doxylamine	100,000
METHADONE (MTD200)			
Methadone	200	Doxylamine	65,000

Analytes	Concentration (ng/mL)	Analytes	Concentration (ng/mL)
METHAMPHETAMINE (MET1, 000)			
ρ-Hydroxymethamphetamine	25,000	(±)-3,4-Methylenedioxy-methamphetamine	12,500
D-Methamphetamine	1,000		
L-Methamphetamine	20,000	Mephentermine	50,000
METHAMPHETAMINE (MET500)			
ρ-Hydroxymethamphetamine	12,500	(±)-3,4-Methylenedioxy-methamphetamine	6,250
D-Methamphetamine	500		
L-Methamphetamine	10,000	Mephentermine	25,000
METHAMPHETAMINE (MET300)			
ρ-Hydroxymethamphetamine	7,500	(±)-3,4-Methylenedioxy-methamphetamine	3,750
D-Methamphetamine	300		
L-Methamphetamine	6,000	Mephentermine	15,000
METHYLENEDIOXYMETHAMPHETAMINE (MDMA1, 000) Ecstasy			
(±) 3,4-Methylenedioxy methamphetamine HCl	1,000	3,4-Methylenedioxyethyl-amphetamine	600
(±)3,4-Methylenedioxyampheta-mine HCl	6,000		
METHYLENEDIOXYMETHAMPHETAMINE (MDMA500) Ecstasy			
(±)3,4-Methylenedioxy-methamphetamine HCl	500	3,4-Methylenedioxyethyl-amphetamine	300
(±) 3,4-Methylenedioxy-amphetamine HCl	3,000		
MORPHINE (MOP 300)			
Codeine	200	Norcodeine	6,000
Levorphanol	1,500	Normorphone	50,000
Morphine-3-β-D-Glucuronide	800	Oxycodone	30,000
Ethylmorphine	6,000	Oxymorphone	50,000
Hydrocodone	50,000	Procaine	15,000
Hydromorphone	3,000	Thebaine	6,000
6-Monoacetylmorphine	300	Morphine	300
MORPHINE (MOP 100)			
Codeine	80	Norcodeine	2,000
Levorphanol	500	Normorphone	20,000
Morphine-3-β-D-Glucuronide	300	Oxycodone	10,000
Ethylmorphine	2,000	Oxymorphone	20,000
Hydrocodone	20,000	Procaine	5,000
Hydromorphone	1,000	Thebaine	2,000
6-Monoacetylmorphine	200	Morphine	100
Methaqualone (MQL 300)			
Methaqualone	300		
MORPHINE/OPIATE (OPI 2,000)			
Codeine	2,000	Morphine	2,000
Ethylmorphine	3,000	Norcodeine	25,000
Hydrocodone	50,000	Normorphone	50,000
Hydromorphone	15,000	Oxycodone	25,000
Levorphanol	25,000	Oxymorphone	25,000
6-Monoacetylmorphine	3,000	Procaine	50,000
Morphine 3-β-D-glucuronide	2,000	Thebaine	25,000
PHENCYCLIDINE (PCP)			
Phencyclidine	25	4-Hydroxyphencyclidine	12,500
PROPOXYPHENE (PPX)			
D-Propoxyphene	300	D-Norpropoxyphene	300
TRICYCLIC ANTIDEPRESSANTS (TCA)			
Nortriptyline	1,000	Imipramine	400

Analytes	Concentration (ng/mL)	Analytes	Concentration (ng/mL)
Nordoxepine	500	Clomipramine	50,000
Trimipramine	3,000	Doxepine	2,000
Amitriptyline	1,500	Maprotiline	2,000
Promazine	3,000	Promethazine	50,000
Desipramine	200	Perphenazine	50,000
Cyclobenzaprine	2,000	Dithiaden	10,000
Tramadol (TML)			
n-Desmethyl-cis-tramadol	200	o-Desmethyl-cis-tramadol	10,000
Cis-tramadol	100	Phencyclidine	100,000
Procyclidine	100,000	d,l-O-Desmethyl venlafaxine	50,000
KETAMINE (KET1, 000)			
Ketamine	1,000	Benzphetamine	25,000
Dextromethorphan	2,000	(+) Chlorpheniramine	25,000
Methoxyphenamine	25,000	Clonidine	100,000
d-Norpropoxyphene	25,000	EDDP	50,000
Promazine	25,000	4-Hydroxyphencyclidine	50,000
Promethazine	25,000	Levorphanol	50,000
Pentazocine	25,000	MDE	50,000
Phencyclidine	25,000	Meperidine	25,000
Tetrahydrozoline	500	d-Methamphetamine	50,000
Mephentermine	25,000	l-Methamphetamine	50,000
(1R, 2S) - (-)-Ephedrine	100,000	3,4-Methylenedioxy-methamphetamine (MDMA)	100,000
Disopyramide	25,000	Thioridazine	50,000
KETAMINE (KET500)			
Ketamine	500	Benzphetamine	12,500
Dextromethorphan	1,000	(+) Chlorpheniramine	12,500
Methoxyphenamine	12,500	Clonidine	50,000
d-Norpropoxyphene	12,500	EDDP	25,000
Promazine	12,500	4-Hydroxyphencyclidine	25,000
Promethazine	12,500	Levorphanol	25,000
Pentazocine	12,500	MDE	25,000
Phencyclidine	12,500	Meperidine	12,500
Tetrahydrozoline	250	d-Methamphetamine	25,000
Mephentermine	12,500	l-Methamphetamine	25,000
(1R, 2S) - (-)-Ephedrine	50,000	3,4-Methylenedioxy-methamphetamine (MDMA)	50,000
Disopyramide	12,500	Thioridazine	25,000
KETAMINE (KET300)			
Ketamine	300	Benzphetamine	6,250
Dextromethorphan	600	(+) Chlorpheniramine	6,250
Methoxyphenamine	6,250	Clonidine	30,000
d-Norpropoxyphene	6,250	EDDP	15,000
Promazine	6,250	4-Hydroxyphencyclidine	15,000
Promethazine	6,250	Levorphanol	15,000
Pentazocine	6,250	MDE	15,000
Phencyclidine	6,250	Meperidine	6,250
Tetrahydrozoline	150	d-Methamphetamine	15,000
Mephentermine	6,250	l-Methamphetamine	15,000
(1R, 2S) - (-)-Ephedrine	30,000	3,4-Methylenedioxy-methamphetamine (MDMA)	30,000
Disopyramide	6,250	Thioridazine	15,000
KETAMINE (KET100)			
Ketamine	100	Benzphetamine	2,000
Dextromethorphan	200	(+) Chlorpheniramine	2,000

Analytes	Concentration (ng/mL)	Analytes	Concentration (ng/mL)
Methoxyphenamine	2,000	Clonidine	10,000
d-Norpropoxyphene	2,000	EDDP	5,000
Promazine	2,000	4-Hydroxyphencyclidine	5,000
Promethazine	2,000	Levorphanol	5,000
Pentazocine	2,000	MDE	5,000
Phencyclidine	2,000	Meperidine	2,000
Tetrahydrozoline	50	d-Methamphetamine	5,000
Mephentermine	2,000	l-Methamphetamine	5,000
(1R, 2S) - (-)-Ephedrine	10,000	Thioridazine	5,000
Disopyramide	2,000	3,4-Methylenedioxyamphetamine (MDMA)	10,000
Oxycodone (OXY100)			
Oxycodone	100	Hydromorphone	50,000
Oxymorphone	300	Naloxone	25,000
Levorphanol	50,000	Naltrexone	25,000
Hydrocodone	25,000		
Cotinine (COT 200)			
(-)-Cotinine	200	(-)-Nicotine	5,000
Cotinine (COT 100)			
(-)-Cotinine	100	(-)-Nicotine	2,500
2-Ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP300)			
2-Ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP)			300
2-Ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP100)			
2-Ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP)			100
Fentanyl (FYL20)			
Alfentanyl	600,000	Buspirone	15,000
Fenfluramine	50,000	Fentanyl	100
Norfentanyl	20	Sufentanyl	50,000
Fentanyl (FYL10)			
Alfentanyl	300,000	Buspirone	8,000
Fenfluramine	25,000	Fentanyl	50
Norfentanyl	10	Sufentanyl	25,000
Synthetic Marijuana (K2-50)			
JWH-018 5-Pentanoic acid	50	JWH-073 4-butanoic acid	50
JWH-018 4-Hydroxypentyl	400	JWH-018 5-Hydroxypentyl	500
JWH-073 4-Hydroxybutyl	500		
Synthetic Marijuana (K2-30)			
JWH-018 5-Pentanoic acid	30	JWH-073 4-butanoic acid	30
JWH-018 4-Hydroxypentyl	250	JWH-018 5-Hydroxypentyl	300
JWH-073 4-Hydroxybutyl	300		
6-mono-aceto-morphine (6-MAM)			
6-Monoacetylmorphine	10	Morphine	100,000
(±) 3, 4-Methylenedioxyamphetamine (MDA 500)			
(±) 3, 4-Methylenedioxyamphetamine	500	Methoxyphenamine	5,000
		D-Amphetamine	2,000
D,L-Amphetamine sulfate	400	Phentermine	2,000
L-Amphetamine	30,000	Maprotiline	100,000

EFFECT OF URINARY SPECIFIC GRAVITY

Fifteen (15) urine samples of normal, high, and low specific gravity ranges (1.005-1.045) were spiked with drugs at 50% below and 50% above cut-off levels respectively. The CLEARTEST® Multi Drug 6 was tested in duplicate using fifteen drug-free urine and spiked urine samples. The results demonstrate that varying ranges of urinary specific gravity do not affect the test results.

EFFECT OF URINARY PH

The pH of an aliquoted negative urine pool was adjusted to a pH range of 5 to 9 in 1 pH unit increments and spiked with drugs at 50% below and 50% above cut-off levels. The spiked, pH-adjusted urine was tested with the CLEARTEST® Multi Drug 6. The results demonstrate that varying ranges of pH do not interfere with the performance of the test.

CROSS-REACTIVITY

A study was conducted to determine the cross-reactivity of the test with compounds in either drug-free urine or drug positive urine containing above related calibrator substances. The following compounds show no cross-reactivity when tested with the CLEARTEST® Multi Drug 6 at a concentration of 100 µg/mL.

NON CROSS-REACTING COMPOUNDS

Acetophenetidin	Cortisone	Zomepirac	d-Pseudoephedrine
N-Acetylprocainamide	Creatinine	Ketoprofen	Quinidine
Acetylsalicylic acid	Deoxycorticosterone	Labetalol	Quinine
Aminopyrine	Dextromethorphan	Loperamide	Salicylic acid
Amoxicillin	Diclofenac	Meprobamate	Serotonin
Ampicillin	Diflunisal	Methoxyphenamine	Sulfamethazine
l-Ascorbic acid	Digoxin	Methylphenidate	Sulindac
Apomorphine	Diphenhydramine	Nalidixic acid	Tetracycline
Aspartame	Ethyl-p-amino-benzoate	Naproxen	Tetrahydrocortisone,
Atropine	β-Estradiol	Niacinamide	3-acetate
Benzilic acid	Estrone-3-sulfate	Nifedipine	Tetrahydrocortisone
Benzoic acid	Erythromycin	Norethindrone	Tetrahydrozoline
Bilirubin	Fenoprofen	Noscapine	Thiamine
d,l-Brompheniramine	Furosemide	d,l-Octopamine	Thioridazine
Caffeine	Gentisic acid	Oxalic acid	d,l-Tyrosine
Cannabidiol	Hemoglobin	Oxolinic acid	Tolbutamide
Chloral hydrate	Hydralazine	Oxymetazoline	Triamterene
Chloramphenicol	Hydrochlorothiazide	Papaverine	Trifluoperazine
Chlorothiazide	Hydrocortisone	Penicillin-G	Trimethoprim
d,l-Chlorpheniramine	o-Hydroxyhippuric acid	Perphenazine	d,l-Tryptophan
Chlorpromazine	3-Hydroxytyramine	Phenelzine	Uric acid
Cholesterol	d,l-Isoproterenol	Prednisone	Verapamil
Clonidine	Isoxsuprine	d,l-Propranolol	

BIBLIOGRAPHY

- Hawks RL, CN Chiang. Urine Testing for Drugs of Abuse. National Institute for Drug Abuse (NIDA), Research Monograph 73, 1986.
- Tietz NW. Textbook of Clinical Chemistry. W.B. Saunders Company. 1986; 1735.
- Stewart DJ, Inaba T, Lucassen M, Kalow W. Clin. Pharmacol. Ther. April 1979; 25 ed: 464, 264-8.
- Ambre J. J. Anal. Toxicol. 1985; 9:241.
- Winger, Gail, A Handbook of Drug and Alcohol Abuse, Third Edition, Oxford Press, 1992, page 146.
- Robert DeCresce. Drug Testing in the workplace, 1989 page 114.
- Glass, IB. The International Handbook of Addiction Behavior. Routledge Publishing, New York, NY. 1991; 216
- B. Cody, J.T., "Specimen Adulteration in drug urinalysis. Forensic Sci. Rev., 1990, 2:63.
- C. Tsai, S.C. et al., J. Anal. Toxicol. 1998; 22 (6): 474
- Baselt RC. Disposition of Toxic Drugs and Chemicals in Man. 6th Ed. Biomedical Publ., Foster City, CA 2002.
- Hardman JG, Limbird LE. Goodman and Gilman's: The Pharmacological Basis for Therapeutics. 10th Edition. McGraw Hill Medical Publishing, 2001; 208-209.
- Cumming, E. (22 April 2010). „Mephedrone: Chemistry lessons“. London: The Daily Telegraph. Retrieved 2010-09-14.
- „Drugs crackdown hailed a success“. BBC News. 8 March 2010. Retrieved 2010-03-31.
- Kihara, Rhiannon; Day, Edward (May 2014). „Transient psychotic episodes following recreational use of NRG-3“. Progress in Neurology and Psychiatry 18 (3): 14–18. doi:10.1002/pnp.331. Retrieved 22 March 2015.
- Schifano, F.; Albanese, A.; Fergus, S.; Stair, J. L.; Deluca, P.; Corazza, O.; Davey, Z.; Corkery, J.; Siemann, H.; Scherbaum, N.; Farre', M.; Torrens, M.; Demetrovics, Z.; Ghodse, A. H.; Psychonaut Web, M.; Rednet Research, G. (2010). „Mephedrone (4-methylmethcathinone; „meow meow“): chemical, pharmacological and clinical issues“. Psychopharmacology 214 (3):593–602. doi:10.1007/s00213-010-2070-x.ISSN 0033-3158. PMID 21072502.
- „Assessment of Zopiclone“ (PDF).World Health Organization. Essential Medicines and Health Products World Health Organization. p.9 (Section 5. Pharmacokinetics). Retrieved 5 December 2015.
- Kratzsch C, Tenberken O, Peters FT et al. Screening, library-assisted identification, and validated quantification of 23 benzodiazepines, flumazenil, zaleplon, zolpidem, and zopiclone in plasma by liquid chromatography/mass spectrometry with atmospheric pressure chemical ionization. J. Mass Spec. 39: 856-872, 2004.
- Gustavsen I, Al-Sammurraie M, Mørland J, Bramness JG. Impairment related to blood drug concentrations of zopiclone and zolpidem compared with alcohol in apprehended drivers. Accid. Anal. Prev. 41: 462-466, 2009.
- R. Baselt, Disposition of Toxic Drugs and Chemicals i Man, 8th edition, Biomedical Publications, Foster City, CA, 2008, pp. 1677-1679.
- Calkins RF, Aktan GB, Hussain KL (1995). „Methcathinone: the next illicit stimulant epidemic?“. Journal of Psychoactive Drugs. 27 (3): 277–85. doi:10.1080/02791072.1995.10472472. PMID 8594170.
- Methcathinone, <https://en.wikipedia.org/wiki/Methcathinone>.
- Bersani, F. S.; Corazza, O.; Simonato, P.; Mylokosta, A.; Levari, E.; Lovaste, R.; Schifano, F. (2013). „Drops of madness? Recreational misuse of tropicamide collyrium; early warning alerts from Russia and Italy“. General Hospital Psychiatry 35 (5):571–3. Baselt RC. Disposition of Toxic Drugs and Chemicals in Man. 2nd Ed. Biomedical Publ., Davis, CA. 1982; 488
- Malenka RC, Nestler EJ, Hyman SE (2009). „Chapter 15: Reinforcement and Addictive Disorders“. In Sydor A, Brown RY. Molecular Neuropharmacology: A Foundation for Clinical Neuroscience (2nd ed.). New York: McGraw-Hill Medical. p. 375. ISBN 9780071481274.
- American Psychiatric Association (2013). „Substance-Related and Addictive Disorders“. American Psychiatric Publishing. pp. 1–2. Retrieved 10 July 2015.
- Juliano LM, Griffiths RR (2004). „A critical review of caffeine withdrawal: empirical validation of symptoms and signs, incidence, severity, and associated features“. Psychopharmacology (Berl.) 176 (1):1–29. doi:10.1007/s00213-004-2000-x. PMID 15448977. Archived from the original on 29 January 2012.
- Arnaud MJ. Pharmacokinetics and metabolism of natural methylxanthines in animal and man. Handb Exp Pharmacol 2011; 200:33-91.
- Jeukendrup AE, Randell R-Fat burners: nutrition supplements that increase fat metabolism. Obes Rev 2011; 193:1-24.
- Cumming, E. (22 April 2010). „Mephedrone: Chemistry lessons“. London: The Daily Telegraph. Retrieved 2010-09-14.
- „Drugs crackdown hailed a success“. BBC News. 8 March 2010. Retrieved 2010-03-31.
- Kihara, Rhiannon; Day, Edward (May 2014). „Transient psychotic episodes following recreational use of NRG-3“. Progress in Neurology and Psychiatry 18 (3): 14–18. doi:10.1002/pnp.331. Retrieved 22 March 2015.
- Schifano, F.; Albanese, A.; Fergus, S.; Stair, J. L.; Deluca, P.; Corazza, O.; Davey, Z.; Corkery, J.; Siemann, H.; Scherbaum, N.; Farre', M.; Torrens, M.; Demetrovics, Z.; Ghodse, A. H.; Psychonaut Web, M.; Rednet Research, G. (2010). „Mephedrone (4-methylmethcathinone; „meow meow“): chemical, pharmacological and clinical issues“. Psychopharmacology 214 (3):593–602. doi:10.1007/s00213-010-2070-x.ISSN 0033-3158. PMID 21072502.
- Work Group on Panic Disorder (January 2009). APA Practice Guideline for the Treatment of Patients With Panic Disorder (2nd ed.).
- „FDA approved labeling for Xanax revision 08/23/2011“ (PDF). Federal Drug Administration. 2011-08-23. p. 4. Retrieved 2011-09-14.
- „Xanax XR (Alprazolam) Clinical Pharmacology – Prescription Drugs and Medications“. RxList. First DataBank. July 2008.
- Frampton, JE (September 2014). „Pregabalin: a review of its use in adults with generalized anxiety disorder.“ CNS Drugs 28 (9): 835–54.
- „Pregabalin“. The American Society of Health-System Pharmacists. Retrieved Oct 23, 2015.
- D.R. Guay. Pregabalin in neuropathic pain: a more “pharmaceutically elegant” gabapentin? Am. J. Geriatr. Pharmacother. 3: 274–287 (2005).

- „Summary of product characteristics“ (PDF). European Medicines Agency. 6 March 2013. Retrieved 6 May 2013.
- „Detection times of Pregabalin in urine after illicit use: when should a positive specimen be considered a new intake?“ Ther Drug Monit. 2013 Feb;35(1):137-40.
- Karniol IG, Shirakawa I, Takahashi RN, Knobel E, Musty RE (1975). „Effects of delta9-tetrahydrocannabinol and cannabiol in man“. Pharmacology. 13 (6): 502–12.
- Andre, Christelle M.; Hausman, Jean-Francois; Guerriero, Gea (2016-02-04). „Cannabis sativa: The Plant of the Thousand and One Molecules“. Frontiers in Plant Science. 7: 19
- McCallum ND, Yagen B, Levy S, Mechoulam R (May 1975). „Cannabinol: a rapidly formed metabolite of delta-1- and delta-6-tetrahydrocannabinol“. Experientia. 31 (5): 520–1
- Mahadevan A, Siegel C, Martin BR, Abood ME, Beletskaya I, Razdan RK (October 2000). „Novel cannabiol probes for CB1 and CB2 cannabinoid receptors“. Journal of Medicinal Chemistry. 43 (20): 3778–85.
- Petitot F, Jeantaud B, Reibaud M, Imperato A, Dubroeuq MC (1998).
- „Gabapentin“. The American Society of Health-System Pharmacists. Archived from the original on 8 September 2017. Retrieved 23 Oct 2015.
- Wijemanne, S; Jankovic, J (June 2015). „Restless legs syndrome: clinical presentation diagnosis and treatment“. Sleep medicine. 16 (6): 678–90.
- „Suicidal Behavior and Ideation and Antiepileptic Drugs“. U.S. Food and Drug Administration (FDA). Archived from the original on 1 August 2009.
- „Neurontin packaging insert“ U.S. Food and Drug Administration (FDA). 1 May 2009. Archived from the original on 14 December 2010. Retrieved 16 July 2010.
- Agarwal P, Griffith A, Costantino HR, Vaish N (2010). „Gabapentin enacarbil - clinical efficacy in restless legs syndrome“. Neuropsychiatr Dis Treat. 6: 151–8.
- Calandre EP, Rico-Villademoros F, Slim M (2016). „Alpha2delta ligands, gabapentin, pregabalin and mirogabalin: a review of their clinical pharmacology and therapeutic use“. Expert Rev Neurother. 16 (11): 1263–1277.
- Calandre EP, Rico-Villademoros F, Slim M (2016). „Alpha2delta ligands, gabapentin, pregabalin and mirogabalin: a review of their clinical pharmacology and therapeutic use“. Expert Rev Neurother. 16 (11): 1263–1277.
- Bockbrader HN, Wesche D, Miller R, Chapel S, Janiczek N, Burger P (2010). „A comparison of the pharmacokinetics and pharmacodynamics of pregabalin and gabapentin“. Clin Pharmacokinet. 49(10): 661–9.
- Cundy KC, Sastry S, Luo W, Zou J, Moors TL, Canafax DM (2008). „Clinical pharmacokinetics of XP13512, a novel transported prodrug of gabapentin“. J Clin Pharmacol. 48 (12): 1378–88.
- Schatzberg, AF; Nemeroff, CB, eds. (2009). Textbook of Psychopharmacology (4th ed.). Washington, D.C.: American Psychiatric Publishing. ISBN 978-1-58562-309-9.
- Amalia Scotto di Telia, Ciro di Nunzio, Pietrantonio Ric (1986). Determination of Trazodone and Its Metabolite, m-CPP, in Serum and Urine by HPLC, 233-235.
- Jauch R, Kopitar Z, Prox A, Zimmer A, Zimmer (1976). „[Pharmacokinetics and metabolism of trazodone in man

Index of symbols

REF	Item number	!	Temperature limitation
	Read instructions for use		Batch number
	In-vitro-Diagnostic		Expiry date
	Manufacturer		Content sufficient for <n> tests
	Dangerous substances		Single use
	Protect from heat and sunlight		Attention
	Protect from moisture		
	Do not use, if package is damaged		
	CE marked according to IVD directive 98/79/EG		

ORDER INFORMATION

CLEARTEST® Multi Drug Methadon

10 Testcassettes
5 Testkassettes

C3 23010
 C3 23015

CLEARTEST® Multi Drug Methamphetamin

10 Testcassettes
5 Testcassettes

C3 23000
 C3 23005

CLEARTEST® DIAGNOSTIK

DRUG TESTS

Available drug tests


Drug type	Abbreviation	Cut-off level	PZN	VE	REF
Amphetamin	AMP	1000 ng/ml	09746066	1 Test	C3 11130-1
			01714380	20 Test	C3 11130
Benzodiazepine	BZD	300 ng/ml	09746072	1 Test	C3 18030-1
			01714411	20 Test	C3 18030
Buprenorphin	BUP	10 ng/ml	09746089	1 Test	C3 19093-1
			01714463	20 Test	C3 19093
Extasy	MDMA	500 ng/ml	10628998	1 Test	C3 19070-1
			01714500	20 Test	C3 19070
Kokain	COC	300 ng/ml	09746095	1 Test	C3 12020-1
			01714517	20 Test	C3 12020
Methadon	MTD	300 ng/ml	09746103	1 Test	C3 19030-1
			01714552	20 Test	C3 19030
Methamphetamin	MET	1000 ng/ml	09746126	1 Test	C3 11330-1
			01714569	20 Test	C3 11330
Morphin	MOR/MOP	300 ng/ml	09746132	1 Test	C3 11230-1
			01714606	20 Test	C3 11230
Spice-Synth. Cannabinoide	SYN CAN	50 ng/ml	10629029	1 Test	C3 14090-1
			10629012	20 Test	C3 14090
Tetrahydrocannabinol	THC	50 ng/ml	09746250	1 Test	C3 13030-1
			01714687	20 Test	C3 13030

ORDER HOTLINE 0281 95283-0 | ANY QUESTIONS? Tel. +49 281 95283-558 · ivd@servoprax.de · www.servoprax.de



Created on 2022-04-26

1-C3 23000ff-222-2-0001.1-2203

 servoprax GmbH
Am Marienbusch 9, 46485 Wesel, Germany
Tel. +49 281 95283-558
ivd@servoprax.de, www.servoprax.de

 **CLEARTEST®** DIAGNOSTIK